



1. **PRODUCT NAME**

Muzziball™

- sensory ball with lights and sounds activated by movement and stimulating activity in the user

2. **Product code**

V22024

3. **Colour**

- Transparent surface
- Multi-colored lights

4. **Brief description**

Muzziball is a plug-and-play, user-operated health-tech sensory ball that produces AI-based personalized, interactive light and audio experiences for all ages

Muzziball is activated by movement and operated through our user-friendly App, which offers a variety of programs developed by specialists and tailored to the needs and preferences of the user with a monthly application subscription and monthly content-upgrades

5. **Content**

1x Muzziball
1x USB-C charger

6. **Muzziball stimulations**

- Touch
- Sight
- Sounds
- Cognitive
- Emotional

7. **Best Use**

Standing or sitting, hold with both hands (assist if this is not possible)

Not to be thrown on the floor or into wall/ceiling.

Move the ball from side to side, up and down, spinning around, exploring what happens when the position of the ball changes, tap/hit the ball

The ball must be kept in motion for the sounds to be audible, this gives the user control over the volume. User-programs can be changed instantly in the Muzziball-app

See www.muzziball.com for videos for ideas and activities

8. **Compatible with Muzzibal products**

- Compatible with the Muzziball APP (OS/Android)
- Compatible with the Muzziball TABLET (Android)

9. **Starting up**

- Download the Muzziball-app (OS/Android)
- Take product out of box.
- Check item has not been damaged in transit in any way before use (call MUZZIBALL if there are any problems)
- Turn the Muzziball on in the app
- Select program
- PLAY

10. **Detailed description**

This item has 15 different programs developed by specialists with tactile, moveable and visual activities which facilitate use of a range of fine finger movements, larger

movements, emotional and cognitive stimuli. As the different elements are utilised, the client will experience different audible and visual input.

11. Safety

- This item should initially be used under supervision.
- Don't intentionally throw on the floor/into walls, ceilings or at other people.
- If the item is damaged, please return it to MUZZIBALL.

12. Technical specifications

- Muzziball (medium) 30cm diameter,850 grams
- Muzziball /large) 50 cm diameter, 850 grams
- Skin-friendly, recyclable, impact-proof, easy-to-clean PETG
- Muzziball HW

13. Installation

- Download the Muzzi-APP (OS/Android)
- Plug&play (ON)
- Ensure that anyone interacting with the Muzziball has enough space to move and enjoy the activities.

14. Care and maintenance

- Initially use under supervision.
- Inspect Muzziball regularly for wear and tear.
- Check Muzziball for damage , if any please contact MUZZIBALL
- Wipe with a damp cloth and a mild soap/disinfectant solution. Dry before use

15. Troubleshooting

- Restart APP
- Restart Muzziball
- Check battery-levels
- Contact MUZZIBALL if any issues

Further copies of these instructions can be downloaded at www.muzziball.com together with some instruction-videos.

We hope you find our instructions valuable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at info@muzziball.com

